



When bonding denture resin to plastic teeth, invest, boil out, wash mold with detergent and flush with clean boiling water. Apply Vita-Sep™ separating media (Item # 3505-09) to mold. Make sure that teeth are clean from any wax residue and detergent film.

Apply Vitacrilic® Bondit™ monomer with a brush to ridge of plastic teeth. Vitacrilic® Bondit™ monomer may also be applied to the surface of the old denture base resin for relines, to eliminate any line between new and old denture base resin.

Refrigerate Bondit™ monomer to prolong shelf life.

Caution: Avoid prolonged handling of uncured mixture and inhalation of monomer vapors. Wear protective gloves when packing resin mixture.

FRICKE DENTAL INTERNATIONAL, INC.
165 Roma Jean Parkway
Streamwood, IL 60107
800-537-4253
Visit our website at www.frickedental.com
MADE IN THE USA